



Product Specifications

HypaSoothe Burn Dressing 10x10

- Sterile gel-soaked burn dressings for minor burns and scalds
- Protects burn injuries and creates a moist wound healing environment
- Non-toxic water based gel formula suitable for 1st and 2nd degree burns
- Non adherent, minimising trauma and skin damage
- Single or pack of 5 available
- Dressing dimensions: 10cm x 10cm

The **HypaSoothe Burn Dressing** is a sterile gel soaked burn dressing ideal for the first aid treatment of minor burn injuries. The nonwoven dressing is soaked in a water based, water soluble gel formula that moistens, cools and soothes burns. The dressings are suitable for use on 1st and 2nd degree burns and scalds. They are sterile and bacteriostatic, ensuring a contamination-free moist wound healing environment. Non-adherent and latex-free. Usage: - Cool the burn with water - Remove the dressing from packaging and apply directly to the burn injury - Secure the dressing with a non-absorbent bandage Caution: - Do not reuse - For external use only - Do not use if packaging is broken - Do not treat a burn with butter, grease, dry dressings, ointments or salves The HypaSoothe Burn Dressing is available in the following sizes: - 10cm x 10cm - 20cm x 20cm



HypaSoothe
Burn Dressing
10x10cm

Sterile Hydrogel Impregnated
Pad Dressing For Burns

First and second degree burns and scalds.
Minimises trauma and skin damage.
Asses most wound healing moisture, cools and soothes.
Non adherent & non toxic.

Effective treatment for burn injuries

The image shows the product packaging on the left and three sequential photographs on the right. The top photo shows a hand holding the dressing out of its pouch. The middle photo shows the dressing being placed over a burn on a hand. The bottom photo shows the dressing fully applied and secured with a white bandage.